



Neurodevelopmental Support Systems: Dandle WRAP™

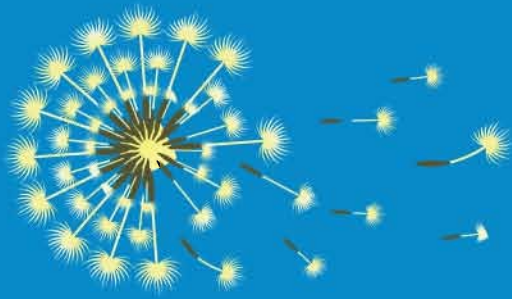

- Upper and lower extremities can be adjusted independently of each other and may be individualized based on baby's needs.
- Micro-fasteners provide secure support for low tone and difficult to console babies.
- Swaddling is easy and fast for parents and staff.
- Caregivers have an "extra set of hands" during procedures.
- Soft, stretchy organic cotton swaddles the baby in comfort.

Shoulder darts help to wrap the wings under the neck and around the shoulder girdle and help to keep the shoulders depressed.

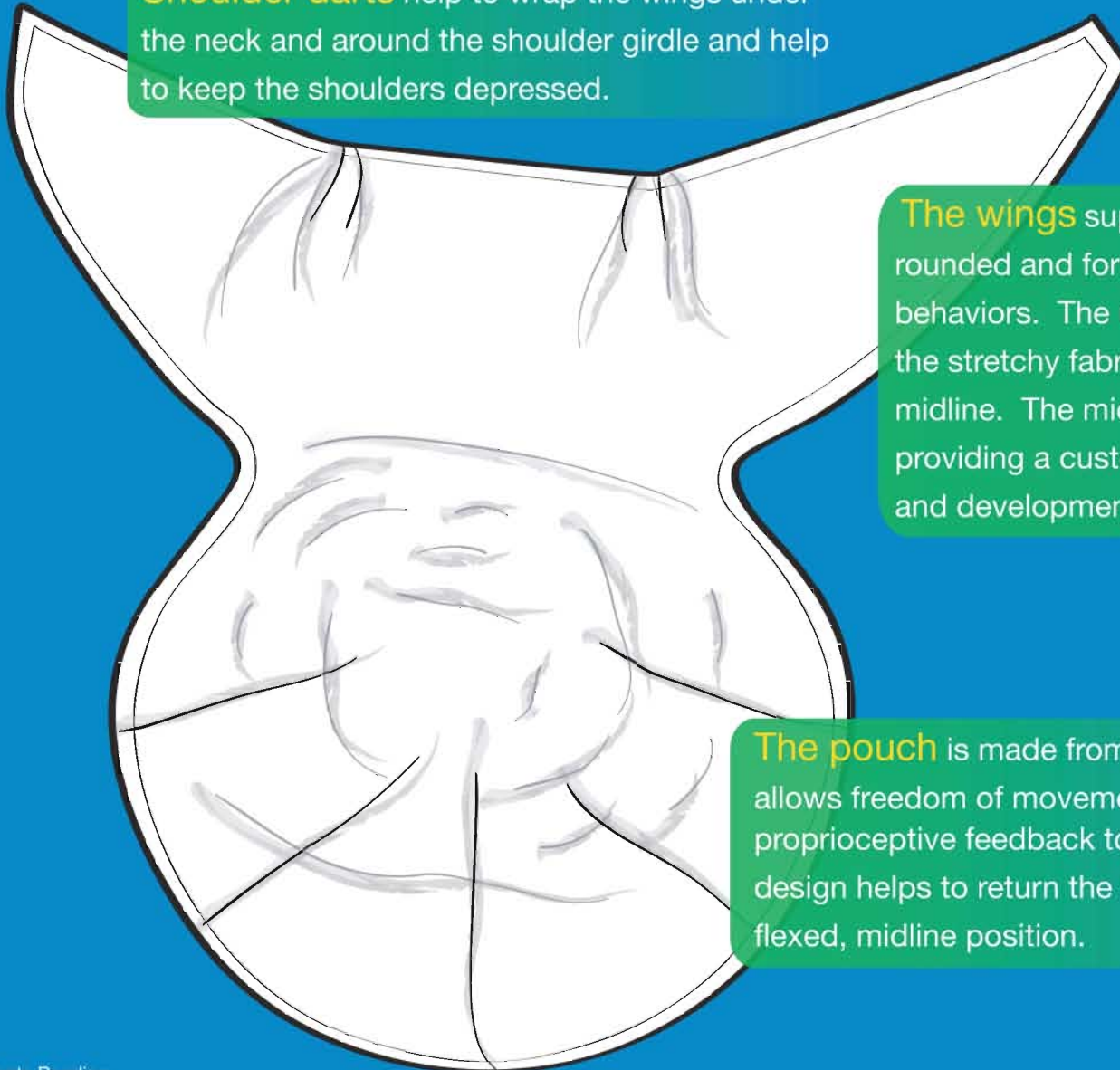

The wings support the shoulders in flexion, keeping them rounded and forward, and encouraging hand to mouth behaviors. The unique wing shape offers secure support while the stretchy fabric allows the baby to move, then return to midline. The micro-fasteners attach easily on the back, providing a customized fit to support the individual medical and developmental needs of the baby.

The pouch is made from a stretchy cotton that allows freedom of movement while providing proprioceptive feedback to the baby. The unique design helps to return the hips, legs and feet to a flexed, midline position.





Neurodevelopmental Support Systems: Dandle WRAP™

Feeding

The WRAP helps facilitate a quality feeding by helping the baby:

- Stay organized and contained during gavage and oral feedings.
- Maintain physiological flexion without restricting movement.
- Keep hands near the face to facilitate grasping and sucking behaviors.
- Move freely and settle into a comfortable position for digestion after feeding.



Feeding

The WRAP helps the caregiver:

- Easily handle and burp the baby without bulky blankets.
- See the head and neck in order to visualize sucking and swallowing.
- Feel respirations through the lightweight fabric.
- Maintain good trunk alignment.
- Have access to the mid section for medically complicated babies.
- Read and respond to baby's cues more easily.



Calming

Adjustable swaddling tension helps to calm irritable babies and allows more stable infants the freedom to move and normalize tone.



Bonding

Soft, lightweight cotton allows parents to "feel" their baby without layers of blankets. Unique design allows parents to easily and confidently swaddle and hold their baby.



Supporting Flexion

The stretchy cotton gives the baby the ability to actively extend and then return to a flexed, midline position which will help normalize movement patterns.



Diaper Changing

Arms and shoulders remain swaddled while pouch is opened for diaper changing. It's like an extra set of hands!



Supporting during Procedures

Wings and pouch can be adjusted independently of each other to meet the baby's medical and developmental needs.



Dandle • LION™
Medical

www.dandleLIONmedical.com